# ENGLISH

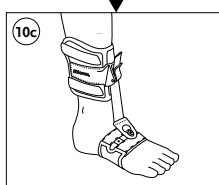
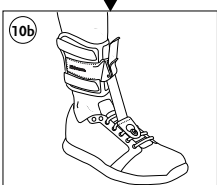
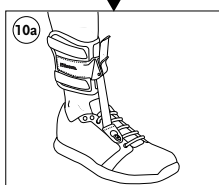
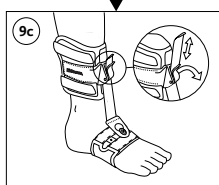
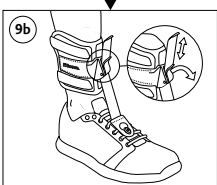
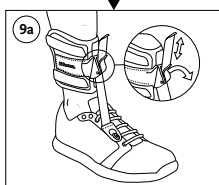
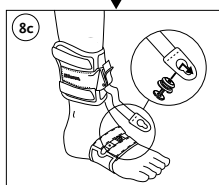
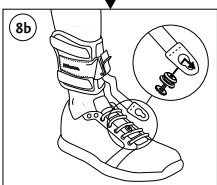
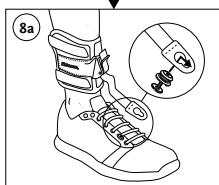
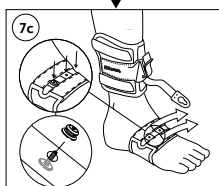
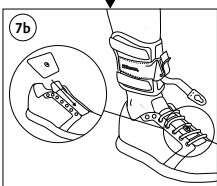
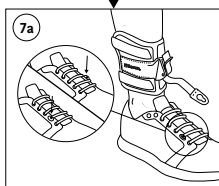
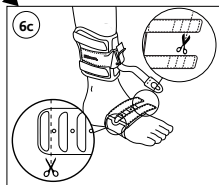
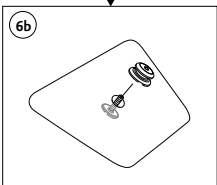
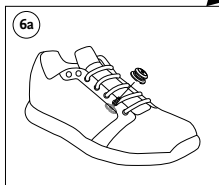
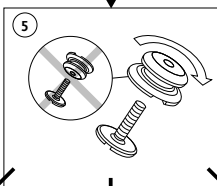
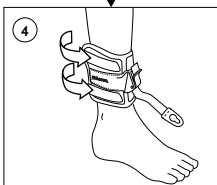
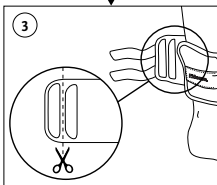
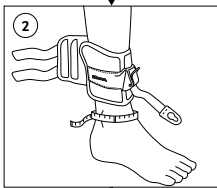
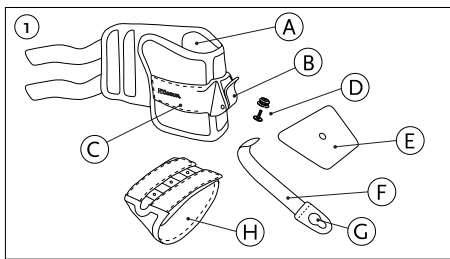
FIGURE 1

- A. Ankle Cuff
- B. Ankle Cuff Buckle
- C. Ankle Clamp
- D. Nut and Bolt
- E. Shoe Inlay
- F. Connection Strap
- G. Connection Strap Keyhole
- H. Optional Foot Wrap

## INDICATIONS FOR USE

Drop foot

## PRECAUTIONS

- Not intended for use by patients with spasticity.
- Never wear the brace over an open wound.
- Users with sensitive skin or those at risk of skin ulcerations and infections, such as in the case of diabetes or peripheral vascular disease, are advised to consult with their practitioner before use of this product.
- Do not over-tighten brace as risk loss of blood flow.
- If the user experiences pain, swelling, sensation changes, or shows signs of insufficient blood flow (turning blue, white, or cold) while using the brace, advised to discontinue use immediately and consult practitioner.

## INITIAL APPLICATION AND ADJUSTMENT

### Donning Ankle Cuff

- Measure ankle circumference with ankle cuff
- Cut ankle cuff to size (Figure 3).
- Secure the ankle cuff (Figure 4).

### Nut and Bolt (Figure 5)

- Ensure a secure and tight fit between nut and bolt.
- Bolt can be screwed with a coin if required.
- Nut fit should be checked periodically and tightened if required.

### Shoe Attachment (for medial or lateral configuration)

- Thread bolt through shoelace eyelet on either side of the shoe (ideally towards the toe-end) for increased medial or lateral control (Figure 6A).
- Fasten bolt with nut securely (Figure 7A).
- Snap keyhole into place over nut (Figure 8A).
- Thread connection strap through ankle cuff buckle, and pull to tighten (Figure 9A).

### Inlay Attachment (for central configuration)

- Thread bolt through inlay, and fasten nut securely (Figure 6B).
- Place inlay between shoelaces and shoe tongue, shortest end of inlay towards toe-end (Figure 7B).
- Snap keyhole into place over nut (Figure 8B).
- Thread connection strap through ankle cuff buckle, and pull to tighten (Figure 9B).

### Optional Foot Wrap Attachment

- Foot wrap and straps can be cut to size if necessary, taking care not to cut over the line of stitching (Figure 6C).
- Thread bolt through foot wrap at either medial, central, or lateral configuration, ensuring nut is securely fastened (Figure 7C).
- Snap keyhole into place over nut (Figure 8C).
- Thread connection strap through ankle cuff buckle, and pull to tighten (Figure 9C).

### Doffing

- When doffing the brace, snap keyhole off the nut.
- Adjustment of the connection strap should not be necessary for subsequent donning.

### WARNING

- Over time the connection strap will lose elasticity, rendering the product less effective at providing dynamic support. The user is advised to replace the strap at regular intervals dependent on their level of activity, and support required.
- The user is advised against donning tight shoes or socks over the Foot Wrap, as this may result in diminished blood flow to/from the foot causing injury.
- For users with lower-leg width in the lowest range, the ankle clamp may not provide an optimal level of hold when donning the device.

### WASHING & CARE

### LIST OF MATERIALS

Nylon, polyethylene, polypropylene, cotton, polyurethane, polyester, silicone (printing).

### WARRANTY

Össur provides a limited 6-month warranty.

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